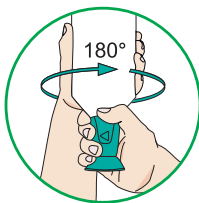
# AUGEN-NOTFALLPLAN

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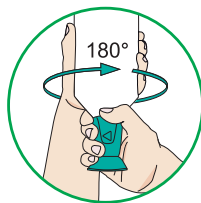
# EYE EMERGENCY PLAN

| Pollutants influence                                                 | Recommended eye wash solution                                                                                                  | Instructions for use                                                                     | Continue until medical attention                                                                                                |
|----------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------|
| <b>Foreign bodies</b><br>such as dirt, dust, metal or wood splinters |  Sodium chloride solution (0,9%)              | Rinse the eyes until the foreign bodies are washed out.                                  |  Rinse with sodium chloride solution (0,9%)  |
| <b>Acid</b>                                                          |  Buffered eye wash solution <b>BioPhos®74</b> | Rinse the eyes with <b>BioPhos®74</b> for approx. 2,5 minutes until the bottle is empty. |  Rinse with sodium chloride solution (0,9%)  |
| <b>Alkali</b>                                                        |  Buffered eye wash solution <b>BioPhos®74</b> | Rinse the eyes with <b>BioPhos®74</b> for approx. 2,5 minutes until the bottle is empty. |  Rinse with sodium chloride solution (0,9%)  |
| <b>Other</b><br>for example caused by solvent, oil                   |  Sodium chloride solution (0,9%)             | Rinse the eyes until the unpleasant feeling is gone.                                     |  Rinse with sodium chloride solution (0,9%) |

Remove the bottle from the eye wash station resp. the eye wash case.

Turn the eye cup 180° in the direction of arrow until the sealing brakes.

Lean your head back and rinse the affected eye consistently.

Lean your head forward before putting the bottle down to protect your clothes from wetness.

**IMPORTANT:** Rinse the eye always consistently and carefully, and thereby keep the eye wide open. Press the bottle slightly while rinsing.

In any case of eye damage, contamination, consult a doctor immediately! Continue rinsing with sodium chloride solution (0,9%) until medical advice is given.